



Please read this information carefully before using me. Do not throw this instruction for use away, it contains important information.

CONTENT OF INSTRUCTIONS

1. USE ME FOR

2. HOW TO USE ME

3. BE CAREFUL WITH ME

4. MY INGREDIENTS

5. HOW TO STORE ME

6. MY CONTACT DETAILS

1. USE ME FOR

Hey you! I'm MAX, the earwax cleaner and together we are going to do this! I'm designed as a hippo that easily flushes out excessive earwax with water and a unique soap formulation making your ears clean and hygienic again.

Earwax, or cerumen, is a self-cleaning agent that your body produces. It collects dirt, bacteria, and other debris. Usually, the wax works its way out of the ears naturally through chewing and other jaw motions.

Earwax can build up and affect your hearing. When earwax reaches this level, it's called impaction.

If you have impaction, you may experience symptoms like:

- aching in the affected ear
- fullness or ringing in the ear
- impaired hearing in the affected ear
- an odor coming from the affected ear
- dizziness
- a cough

Luckily I'm here to help you! I'm based on the same technique used by ear specialists. I gently rinse your ear canal using water enriched with unique soap. A simple method for cleaning ears. I'm an effective solution for regularly cleaning the ear canals at home. Like brushing your teeth, I should be part of your personal health and hygiene.

2. HOW TO USE ME


With only one application every two days, your ears are clean and you can listen to what your mom is saying again!

Repeat the procedure three times on each occasion. My blue soap will last around 30 days. Replace me once my soap has run out. The best time to use me is during your usual shower or bath routine. Do you have an hearing aid? No problem! Use me right before bed.

1. Fill: Squeeze my suction bulb to draw up lukewarm water (max 40°C) The water will mix with the soap contained in my lower part. Using me for the first time? Rinse three times before rinsing your ear.

2. Rinse: Tilt your head, place my nozzle at the entrance to the ear canal and squeeze. Let the soapy solution soften the ear wax and tilt your head back to drain your ear.

3. Dry: When I'm empty you can use me to gently blow air into the ear canal until dry.

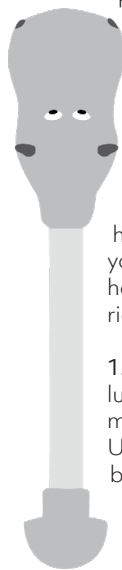

▲ 3. BE CAREFUL WITH ME

It is not recommended to keep using me for more than 30 days consecutively. If complaints aggravate, contact your physician. Do not let other people use me to avoid cross contamination.

I'm intended for application in children, supported by an adult.

Keep away from children below 3 years. Small parts present a choking hazard – keep out of reach of children.

I must not be applied:

- In persons with pre-damage (e.g., perforation) or deformation of the ear drum.
- In persons with pre-damaged ear canal
- In children under 3 years of age.
- In children with ear tubes.
- In children with Otitis (media) or ear infection.
- In children with painful ears.

If your ear canals are blocked, I recommend visiting an ENT specialist. After the ears have been professionally cleaned by a specialist, I recommend flushing them regularly with me to prevent further earwax build-up.

Allergies

Stop using me if there is any allergic reaction. Do not use if you are allergic to one of the ingredients.

Side effects

I'm well tolerated. There are no known side effects, when I'm used according to the instructions for use.

Interference with medications

There is no known negative effect between me and other medications.

4. MY INGREDIENTS

Sodium Tallowate, Glycerin, Aqua (water), Sorbitol, sucrose, PEG-400, Tetrasodium.

5. HOW TO STORE ME

Store at room temperature, (10-25°C). Use me within 8 months after first use.

6. MY CONTACT DETAILS

All enquiries about me should be sent to the distributor. For more information about me, please visit www.kidsner.com.

I'm a medical device, read my instructions for use before use.

Distributor

TheOTCLab

Paulus Potterstraat 20-III
1071 DA Amsterdam
The Netherlands
www.theotclab.com

Manufacturer

TheOTCLab

Paulus Potterstraat 20-III
1071 DA Amsterdam
The Netherlands
www.theotclab.com

Last revision date: July 2018



KD1000TI/18F1