



Please read this information carefully before using me. Do not throw this instruction for use away, it contains important information.

CONTENT OF INSTRUCTIONS

1. USE ME FOR

2. HOW TO USE ME

3. BE CAREFUL WITH ME

4. MY INGREDIENTS

5. HOW TO STORE ME

6. MY CONTACT DETAILS


1. USE ME FOR

Hey you! I'm ALBERT, the anti-allergic rhinitis and together we are going to do this! The bottle contains a natural, non-toxic formulation that effectively treats and prevents allergic rhinitis, such as hay fever, pollen, house dust mites and animal dander. I contain a natural salt solution, which reduces the swelling of the nasal mucosa, thins the mucus and relieves the congestion, allowing you to breathe freely again. The active ingredient provides a protective layer on the nasal mucous membrane, which serves as a barrier against pollen, house mites and other allergens from passing through. Liposomes absorb and inactivate the allergens.

2. HOW TO USE ME

Shake me before use. Hold my bottle upright and insert the tip of the nasal spray bottle into the nostril. With only one or two sprays, the sneeze goes away! I'm fast, safe and effective and I'm going to help you. Spray each nostril 2 to 3 times a day, once or twice, as required. I'm suitable to be used by children of three years and older.

First use: Remove my cap from the bottle. Pump twice or three times until some spray is squirted out of the bottle. If the spray function is blocked (for instance because the nozzle is not being held upright), do not cut off the tip of the bottle. Instead, squeeze a few times to squirt the spray right up into the air.

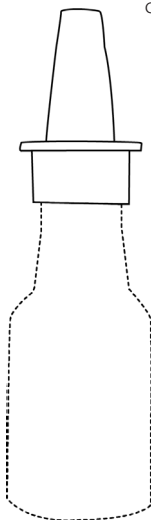

- Blow your nose before use;
- Remove the cap from the bottle;
- Close one nostril with your finger and insert the tip of the nasal spray into the other nostril;
- Spray once or twice;
- Slowly breathe in through the nostril;
- Repeat this procedure for your other nostril;
 - Clean the bottle with a tissue and replace the lid on the nasal spray bottle.

△ 3. BE CAREFUL WITH ME

- Do not use me longer than 30 consecutive days;
- To date, no hypersensitivity reactions have been noted. Hypersensitivity reactions to one of my ingredients, however, may occur. If this happens, stop using me;
- Don't use me without your mom there;
- Don't use me after nasal surgery;
- Don't use me after the expiry date stated on my packaging.

Hypersensitivity

To date, no hypersensitivity reactions have been noted. Hypersensitivity reactions to one of my ingredients, however, may occur. If this happens, stop using me.

Side Effects

To date, no side effects have been noted when using me.



4. MY INGREDIENTS

Amigel, Aqua demi, AquaCacteen, Citric acid solution, Dermosoft 1388 ECO, Nano-Lipobelle H-AECL, Suprasel Fine.

5. HOW TO STORE ME

Store at room temperature (15-25°C) in a dry and dark place in the original container and out of reach of young children. My expiry date is printed on the bottom of the packaging. After opening, do not use longer than 6 weeks.

6. MY CONTACT DETAILS

All enquiries about me should be sent to the distributor. For more information about me, please visit www.kidsner.com.

I'm a medical device, read my instructions for use before use.

Distributor

TheOTCLab
Paulus Potterstraat 20-III
1071 DA Amsterdam
Nederland
www.theotclab.com

Manufacturer

Peters Krizman
AG Schweizergrasse 20
CH-8001 Zurich
Switzerland

Last revision date: July 2018



10ml



KD3000TI+18F1