



Please read this information carefully before using me. Do not throw this instruction for use away, it contains important information.

## CONTENT OF INSTRUCTIONS

1. USE ME FOR
2. HOW TO USE ME
3. PLEASE KEEP IN MIND
4. MY INGREDIENTS
5. WHAT DO I CONTAIN
6. HOW TO STORE ME
7. MY CONTACT DETAILS

### 1. USE ME FOR

Hey you! I'm Max, the earwax remover and together we are going to do this! I'm designed as a hippo that easily flushes out excessive earwax with water and a unique soap formulation making your ears clean and hygienic again.

Earwax, or cerumen, is a self-cleaning agent that your body produces. It collects dirt, bacteria, and other debris. Usually, the wax works its way out of the ears naturally through chewing and other jaw motions.

Earwax can build up and affect your hearing. When earwax reaches this level, it's called impaction.

If you have impaction, you may experience symptoms like:

- aching in the affected ear
- fullness or ringing in the ear
- impaired hearing in the affected ear
- an odor coming from the affected ear
- dizziness
- a cough

Luckily I'm here to help you! I'm based on the same technique used by ear specialists. I gently rinse the ear canal using water enriched with unique soap. A simple method for cleaning ears. I'm an effective solution for regularly cleaning the ear canals at home. Like brushing your teeth, I should be part of the personal health and hygiene.

### 2. HOW TO USE ME

With only one application every two days, ears are clean and

you can listen to what your mom is saying again! Repeat the procedure three times on each occasion. My blue soap will last around 30 days. Replace me once my soap has run out. The best time to use me is during the usual shower or bath routine. Do you have an hearing aid? No problem! Use me right before bed.

Using me for the first time? Rinse three times before rinsing your ear.

1. **Fill:** Squeeze my suction bulb to draw up lukewarm water (max 40°C) The water will mix with the soap contained in my lower part
2. **Rinse:** Tilt the head, place my nozzle at the entrance to the ear canal and squeeze. Let the soapy solution soften the ear wax and tilt the head back to drain your ear
3. **Dry:** When I'm empty you can use me to gently blow air into the ear canal until dry

### ⚠ 3. PLEASE KEEP IN MIND

It is not recommended to keep using me for more than 30 days consecutively. If complaints aggravate, contact your physician. Do not let other people use me to avoid cross contamination. I'm intended for application on children, supported by an adult. Keep out of reach from children below 3 years, small parts present a choking hazard.

#### I must not be applied:

- On persons with pre-damage (e.g., per-foration) or deformation of the ear drum
- On persons with pre-damaged ear canal
- On children under 3 years of age
- On children with ear tubes
- On children with Otitis (media) or ear in-fec-tion
- On children with painful ears

If ear canals are blocked, I recommend visiting an ENT specialist. After the ears have been professionally cleaned by a specialist, I recommend flushing them regularly with me to prevent further earwax build-up.

#### Allergies

Stop using me if there is any allergic reaction. Do not use if you are allergic to one of the ingredients.

#### Side effects

I'm well tolerated. There are no known side effects, when I'm used according to the instructions for use.

#### Interference with medications

There is no known negative effect between me and other medications.

### 4. MY INGREDIENTS

Sodium Tallowate, Glycerin, Aqua (water), Sorbitol, sucrose, PEG-400, Tetrasodium.

#### 5. WHAT DO I CONTAIN

- Applicator
- Instructions for use

#### 6. HOW TO STORE ME

Store me at room temperature, (10-25°C) and out of reach of young children. Use me within 8 months after first use.

#### 7. MY CONTACT DETAILS

We care about your kids. At KIDSNER we love to make everyone a little happier. That's why we have developed dedicated products especially for kids, which can reduce the symptoms of everyday health issues in a fast and effective way. Want to know more? Please visit Kidsner at Kidsner.com

All enquiries about me should be sent to the distributor.

I'm a medical device, read my instructions for use before use.

#### **Manufacturer / Distributor**

TheOTCLab

Fred. Roeskestraat 115

1076 EE Amsterdam

The Netherlands

[www.theotclab.com](http://www.theotclab.com)

Last revision date: January 2022



KD1000TI/22A1