



Carefully read the instructions for use prior to using me, even if you used me before. The information may have been amended according to new research and findings in the time since you opened your last package. I could be obtained without a prescription. In spite of this, **take care** when using me in order to obtain a good result. **Keep** these instructions for use in case you need to consult them again.

CONTENT OF INSTRUCTIONS

1. USE ME FOR
2. HOW TO USE ME
3. PLEASE KEEP IN MIND
4. MY INGREDIENTS
5. WHAT DO I CONTAIN
6. HOW TO STORE ME
7. MY CONTACT DETAILS

1. USE ME FOR

Hey you! I'm Wally the anti-wart remover and together we are going to do this! I'm a medical device for the treatment of common warts. I contain monochloroacetic acid, a substance having a degrading action on the skin. The wart will dissolve, layer after layer. Monochloroacetic acid will make sure that the cells containing the wart virus disappear.

The benefit of me is that dabbing the wart does not hurt, which means that it can also be used for children, when applied correctly by an adult.

I can be used only with common warts

- Primarily occur on hands and feet
- Same colour as the skin
- Circumference varies from less than a millimetre to more than a centimetre
- Several warts usually occur simultaneously
- Contamination often occurs through direct contact, but also through towels or door handles
- Scratching and nail biting increase the risk of contamination



Do not use me with senile warts

- Can come in different sizes and colours
- Average circumference is 0.5 to 1 cm
- Often start as a skin-coloured, yellow-brown, greyish-brown patch. Later on, become bigger and fatter
- Appear as if they are lying on the surface of the skin and are easy to scratch off, but are actually firmly attached to the skin
- They can occur anywhere on the body, except for the palms of the hands and soles of the feet, and the lips. Most likely to occur on the chest, the back, the face, and the neck



2. HOW TO USE ME

Please apply me with utmost care. The solution is caustic. Be extra careful when applying me to body parts that are more difficult to reach. This particularly applies to persons with a limited mobility or in case of disturbed fine motor functions in the system (trembling hands, stiff fingers). Carefully close my bottle after use.

I should not be applied more than once per week. Depending on the size and thickness of the wart, it may be necessary to repeat the treatment with me after one week. Warts are contagious, that is why we do not recommend using me for more than one person.

1. Avoid contact with clothes, textile or table-cloths. Before removing the cap entirely from the vial, apply Vaseline or zinc ointment generously to the healthy skin surrounding the wart

2. I do have a child proof cap that can be opened as follows: Place the bottle on a solid underground that is able to resist the caustic fluid. You can open the bottle by pressing down the cap and turning it anticlockwise simultaneously.

3. Remove the cap from the bottle. Then, apply a small amount of me to the wart using the spatula (do not allow the fluid to drip). Make sure that you only apply me to the wart and do not dab the wart for more than one time.

4. Allow it to dry for a moment and do not cover the wart.

5. Carefully close the bottle after use. The fluid will need a week to dissolve the top layer of the wart, so you will not see an immediate result after application.



3. PLEASE KEEP IN MIND

Do not use me:

- Under the age of 4 years old
- If you are oversensitive to one or more of my ingredients
- In the face (the fluid should never make contact with the eyes or the mucous membrane in the nose or mouth)
- For genital warts or in the genital area
- On birthmarks, water warts, open wounds, moles, naevus, senile (brown) warts, darkly coloured, hairy or strange looking warts
- On red, irritated or infected warts
- Immediately after a surgical removal of warts, freezing of warts (cryotherapy) or a laser treatment. This particularly



applies when the skin is red or painful after the treatment.

In this case you will have to wait for at least 4 weeks, until the redness or inflammation of the skin has fully disappeared. Only then you can start using me

- During pregnancy or while breastfeeding
- Do not use on warts which occur completely or partly underneath the nail

Be extra careful when using me:

- In case of doubts concerning the diagnosis (the change in the skin can not be clearly explained as a wart)
- In case of immune deficiency (e.g. when you are using cytostatic medication)
- In case of diabetes, or other complaints such as vascular diseases, as a result of which wounds will heal less easily.
- In these cases, you may not use me before a doctor has been consulted



⚠ Warnings:

- Only intended for external use
- Use with utmost care. The fluid of me is corrosive
- If the healthy skin makes contact with me, please rinse it with water. If, in spite of these precautions, the skin becomes irritated, it will have to undergo the same treatment as burns
- Contact your general practitioner if your complaints continue to exist
- Properly close the bottle after use and store it in an upright position
- Keep out of reach of young children
- Use by children, the elderly and less able must be supervised by an adult
- Avoid contact with clothes, textile, table-cloths or skin
- Stop using if blisters occur

Side Effects

I may cause some side effects, although not everyone will experience these side effects. The side effects that could occur when using me are: irritated skin, burning sensation, skin redness or oversensitivity reactions. If any of these side effects becomes serious, or if you notice any side effects not listed in these instructions for use, please consult your doctor or pharmacist.

Interference with medications

As far as we know, there are no interactions with any other medications.

4. MY INGREDIENTS


Monochloroacetic acid, purified water.

5. WHAT DO I CONTAIN

- 2ml bottle with applicator
- Instructions for use

6. HOW TO STORE ME

Store me at a temperature of 5-25°C in a dry and dark place in the original container and out of reach of young children.

 My expiry date is stated on the bottom of the packaging. After opening, do not use longer than 6 weeks.

7. KIDSNER CONTACT DETAILS

We care about your kids. At KIDSNER we love to make everyone a little happier. That's why we have developed dedicated products especially for kids, which can reduce the symptoms of everyday health issues in a fast and effective way. Want to know more? Please visit Kidsner at [Kidsner.com](https://www.kidsner.com)

All enquiries about me should be sent to the distributor.

Medical device: Kidsner Wally Wart Treatment is a class IIa Medical Device according to the Directive 93/42/EEC.

Distributor

[TheOTCLab](https://www.theotclub.com)
Fred. Roeskestraat 115
1076 EE Amsterdam
The Netherlands
www.theotclub.com

Manufacturer

[P.K. Benelux B.V.](https://www.pkb-nl.com)
Vluchtoord 17
5406 XP Uden
The Netherlands

Last revision date: January 2022



2ml e  0481